
Instructions For Use

User Instructions

elements
body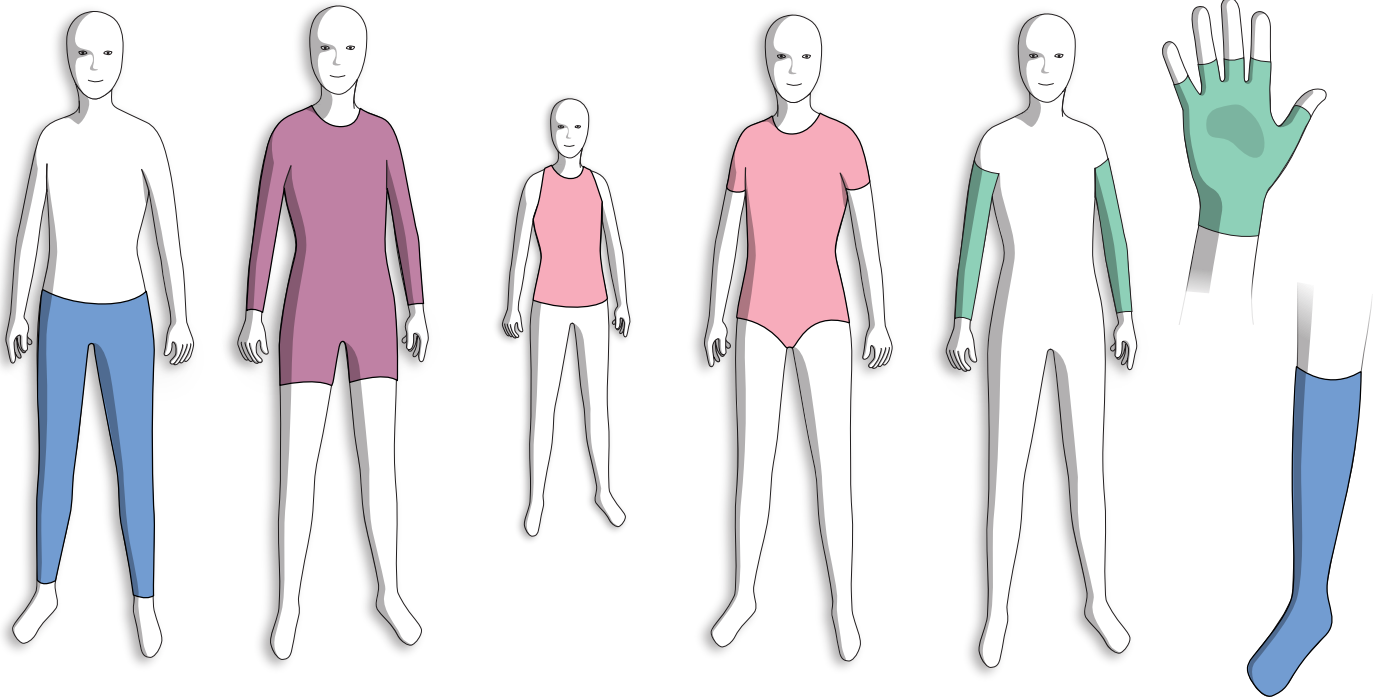

Instructions For Use - User Instructions

EN	Elements Body - Dynamic textile orthosis	p. 4
SV	Elements Body - Dynamisk textil ortos	p. 5
NO	Elements Body - Dynamiske tekstile ortoser	p. 6
DA	Elements Body - Dynamisk tekstil ortose	p. 7
FI	Elements Body - Dynaamiset tekstiiliortoosit	p. 8
PT	Elements Body - Dinamismo dos ortos têxteis	p. 9
ET	Elements Body - Dünaamiline tekstiilist ortoos	p. 10

Product Description

Elements Body is a dynamic textile orthosis, made to measure to meet patient clinical needs.

Intended Use

Elements Body is an orthosis that provides a versatile intervention that can be effective in the management of both high and low tone across a spectrum of neurological conditions. The orthosis can enhance sensory and proprioceptive feedback, fine motor skills and postural control.

This is a made-to-measure product for adults and children that will meet the clinical needs identified by the healthcare professional.

Single patient, multiple use.

Indications

Elements Body is suitable for the following neurological diagnoses presenting with specific symptoms.

- Cerebral Palsy: Hypotonia, Spasticity, Ataxia
- Multiple Sclerosis, Stroke & Acquired Brain Injury: Spasticity, Ataxia
- Muscular Dystrophy, Acquired Spinal Cord Injury and Downs Syndrome: Hypotonia
- Athetosis: Spastic Athetosis

Contraindications

- Osteoporosis
- Osteogenesis imperfecta

Limitations

Use the orthosis initially for short periods of time e.g. 1-2 hours to allow your body to get used to the product. Increase wearing time gradually until 6-8 hours continually wear is achieved.

Note, if the product is worn for more, you may feel over-stimulated and fatigued. The orthosis should not be worn at night while sleeping.

Redness/skin marks can show, usually in the armpit or in the area of an opening or under a zipper. If redness/skin marks are noted, gently massage the area to improve blood circulation. Redness or skin marks should normally disappear after 40 minutes.

If any red skin marks or irritation last more than 8 hours with the product removed, please discontinue use and consult with the prescriber.

It is important to ensure that the integrity of the skin and the garment are intact before application of the orthosis, to prevent damage to the skin.

Limitations may arise when providing Elements Body Orthoses to patients with complex needs. Challenges related to donning and doffing, as well as caregiver comprehension, may affect compliance.

Warnings

If you have a skin condition, circulation disorder, poor temperature regulation, oedema and/or compromised respiratory function, it is important to secure medical clearance before provision. The orthosis must be dry before fitting. Avoid using skin care products immediately prior to application of the orthosis.

Material Specification

Shell fabric: 51% polyamide 32% elastane 17% cotton

Reinforcement fabric: 81% polyamide 19% elastane

Depending on model other materials may be included.

Washing Instruction

Elements Body is made from elasticated material that will benefit from regular washing. We recommend washing every third day to allow material restitution to ensure that it retains its technical properties. Close zippers before washing and use a laundry bag to protect zippers, fasteners etc. Please use a mild, non-biological (free from enzymes) detergent. Do not use fabric softener. It is important that the product is completely dry before use.

Report

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Disposal

Disposal in accordance with local legislation (combustible waste).

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